

SHEEP/GOAT HAIR SAMPLE COLLECTION INSTRUCTIONS

1. Check the Tulsa tag of the animal, and record it on the hair sample envelope.
2. Use bent nose, long nose or needle nose pliers to collect the sample. **REMEMBER:** Cleanse hands and pliers between animal samples to ensure that hairs from different animals are not mixed.
3. We strongly recommend that you collect the hair from the leg, just above the toes. Clean the sample area with a paper towel to remove excess dirt if necessary.
4. Pull a tuft of hair directly away from the skin, **NOT** at an angle. This will allow the hair root to come out with the hair. Take at least three pulls. Clean the sample area with a paper towel to remove excess dirt if necessary.
5. Inspect the hair sample to ensure at least 40 hair follicles.
6. Do **NOT** cut the hair from the animal. The hair **MUST CONTAIN ROOTS** for DNA testing. Avoid touching the roots and make sure the hair is dry.
7. Place hairs in the sample envelope and seal the envelope. Do not put hairs in a plastic bag.
8. Fill out the remaining information lines and signatures on the envelope.
9. Sheep/Goat Hair Sample Checklist
 - Record Tulsa tag number on the envelope
 - Collect hair from the tail switch
 - Obtain at least 40 hairs with follicles
 - Take at least 3 pulls
 - Inspect for follicles—do **NOT** touch follicles
 - Obtain all required information signatures and seal envelope
 - Clean pliers and hands between animals